QMR THERAPY

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*The **Q-Physio** is a newly developed Medical Device which is based on the innovative **QMR technology (Quantum Molecular Resonance)**. Unlike more traditional Radiofrequency devices, **Q-Physio is not based on the hyperthermia (heating) effect**.*

*Due to the peculiar spectrum of multiple frequencies, transferred simultaneously, and its unique way of interacting at cellular level, **the injured tissue is stimulated in such a way that a regenerative process is induced**.*

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*The **QMR (Quantum Molecular Resonance)** is an innovative technology that provide the interaction of molecular bonds that make up tissue with an electric field having a specific spectrum of frequencies transferred simultaneously. The **QMR** is applied in Surgery where is highly appreciated because it's able to reduce the thermal damage at a negligible level, and in Physiotherapy where is capable to induce the **regeneration of an injured tissue through the cellular stimulation**.*

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*The **QMR** in Physiotherapy is not based on a tissue heating effect but on the regeneration of the biological tissue through the particular cell stimulation. This interaction with the cells occurs in a completely different way than the traditional hyperthermia.*

*The effectiveness and capability of the **QMR** technology in physiotherapy is supported by scientific studies that prove its validity.*

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*During the several years of research conducted in collaboration with the **Department of Human Anatomy and Physiology, University of Padua**, it has been discovered that, when the high frequency electrical fields generated by Q-Physio (Rexonage Technology) interact with the muscle cells in culture, the metabolic pathways are being activated and produce the following effects:*

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*A single muscle cell stimulated by the **QMR** undergoes a series of contractions and relaxations, depending on whether the field is turned on or off.*

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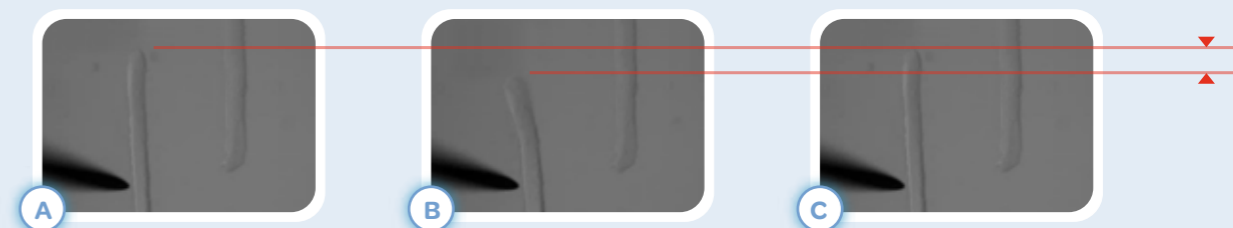
*By using a fluorescence probe was observed that when the **QMR** is activated the membrane potential is depolarizing and then returns to its "resting" potential when the **QMR** is interrupted.*

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
*By using fluorescence probe of the muscle cells stimulated by the **QMR** was observed a release of calcium ions from the sarcoplasmic reticulum, which means that intracellular pathways calcium-dependent has been activated.*

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*The following pictures A,B, C show a single muscle cell being stimulated by the **QMR**. The cell misshapes when the **QMR** electrical field emitted from the tip of a tiny electrode is on (Picture B) and then returns to its original shape when the stimulation is interrupted (Picture C). This stimulation, which is a sort of "cellular massage", has an impact on the different membrane proteins (receptors), intended to detect mechanical type of stress and **induce a cell mitosis process**.*

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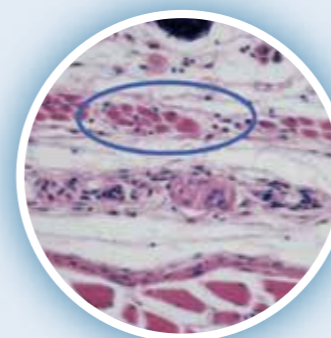*Finally, one of the university studies on the tissue showed that after a course of a **QMR** stimulation the parvocellular infiltrate (black dots) were observed inside the muscle bundles. **This is a result of the regenerative process in progress.***

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COMPARISON OF THE TECHNOLOGIES IN PHYSIOTHERAPY

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The therapy with the Q-Physio does not confine in repairing the lesion with scar type fibrotic tissue, but it rather stimulates the regenerative capabilities of the tissue itself which leads therefore to the regrowth of tissue with same properties and characteristics of the original one.

Ampia gamma di patologie trattabili *Wide range of treated areas and pathologies*

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The therapy with Q-physio is very effective in the treatment of a wide range of pathologies, due to the particular energy emitted with a wide spectrum of frequencies (from 4MHz to 64 MHz), which are simultaneously transmitted to the patient.

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Q-Physio can be safely used on all patients. Exclusion is restricted only to pregnant women and patients bearing pacemaker. The specific frequencies used by Q-Physio allow a safe use even on the treated areas with the prosthesis.

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The patient may undergo the therapy immediately after an injury, since there is no hyperthermia effect. Due to a fast anti-inflammatory and anti-edema action the patient benefits even from the first therapy sessions, of a significant pain decrease. Thanks to the therapy made with Q-Physio the healing is usually stable, permanent and without relapses.

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In order to ensure maximum safety for the patient, Telea has performed a series of experiments in order to ascertain that the stimulation conducted with QMR is totally safe and does not induce any type of cellular damage.

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In the course of the extensive experience gained in the use of QMR technology in physiotherapy, Telea can certainly affirm that it has always proved to be very safe and without side effects.

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The therapeutic action of the Q-Physio is usually accomplished in two phases. Firstly, induces a strong anti-inflammatory and anti-edema effect which leads to a significant decrease of pain. During the second phase, which occurs later in the following weeks until the end of the therapy treatment cycle, the damaged tissue regenerates in a stable and permanent form.

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The treatment of acute pathologies usually gives positive results very quickly. In these cases, a substantial reduction of the pain is obtained after only 2 - 3 sessions.

The patient should, however, continue and conclude the cycle of planned sessions, in order to achieve a complete regeneration of the injured tissues, therefore stabilizing and consolidating the result.

The standard protocol prescribes 2 - 3 sessions per week, although the treatment may also be carried out every day. The duration of each therapeutic session ranges usually from 10 to 25 minutes, depending on the type of pathology and the extension of the treated area.

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Although Q-Physio is very effective in the treatment of chronic and / or degenerative diseases, it can achieve great results in acute diseases and in Sports Medicine for both efficacy and rapidity in the outcome.

Thanks to the QMR technology the injured athlete can be immediately subjected to the therapy, speeding up the period of healing.

Top ranking sports clubs that are using QMR technology in the treatment of injured muscle and / or articular injuries are able to reach a rapid lapse and significantly speed up the resumption of competitive activity.

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With its compact size and light weight, the Q-Physio is well suited to the dynamic Therapist, when it is necessary to work not only at the studio but to travel directly to the patient or follow the sport clubs. The device is easy to carry thanks to the functional trolley and it is easy to use through an intuitive and functional display interface. The setting of the therapy parameters is very quick and effortless.

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*The Q-Physio therapy is executed by employing different accessories developed to exploit to the utmost the **QMR technology**. During the energy delivery, patient feels pleasant warmth that seems to appear from the body. The warmth should never be annoying or unpleasant. It is also possible to combine multiple accessories in the same therapeutic session.*

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Q-Physio can be used using Adhesive Electrodes. Once these electrodes are placed on the area to be treated, they are activated sequentially, following the program set by the Therapist. This method is particularly useful in the cases where the operator has to follow multiple patients simultaneously.

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The handpiece is ergonomic and lightweight. In order to improve the smoothness of the accessory on the injured area the Therapist has to lubricate it by using the neutral cream provided (ref. FR8). At the top of the handpiece a spheroid has to be connected, two sizes are available: one with a diameter of 35 mm and a bigger one with a diameter of 50 mm, created for the treatment of large areas.

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The innovative Q-Touch gloves are capable to deliver energy through the hands of the Therapist without the latter being involved. In this way a perfect combination of manual therapy and stimulation with QMR high frequency electrical fields is achieved.

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info@teleamedical.com
www.teleamedical.com

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Telea Electronic Engineering S.r.l.